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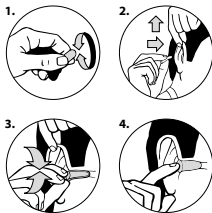
USER INSTRUCTION

Insertion

1. Roll and compress the earplug between the fingers into a smooth, crease-free cylinder.
2. To ensure a proper fit reach hand over head and gently pull ear upward and outwards.
3. Insert compressed, rounded end of the earplug well into the ear canal. The pointed end should go in first.
4. Hold for a few seconds while plug expands. When correctly fitted the earplug should not stick out of ear. Incorrect insertion does not give full protection.

Removal

To avoid damaging the ear canal, remove the earplug by gently twisting and slowly pull the earplug out of the ear canal.

This product is designed to provide adequate protection against harmful noise. Earplugs must be fitted, adjusted and maintained in accordance with the instructions. If instructions are not followed the function and protection may be severely impaired. Earplugs should be worn at all times in noisy surroundings and be fitted prior to entering the noise area. This product may be adversely affected by certain chemicals. Further information should be sought from the manufacturer. These earplugs are disposable, but when not in use store in clean, dry conditions at a temperature of -20° to +50°C.

If you have a skin irritation in your ear canal, please contact your physician before use. In order to reduce the risk of infection it is important that both hands and the earplugs are clean before insertion into ears.

Store out of reach of children. There is a risk of choking if the plug is stuck in the windpipe.

If you are unable to fit these earplugs correctly and comfortably in both ears, contact the manufacturer.

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