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ENG

## USER INSTRUCTIONS

The product consists of two parts; the pill shaped filter and the reusable universal fit ear tip (Image A).

The filter is fitted into the largest supplied ear tip size (1-12 mm). First try inserting this size into the ear canal by following the instructions - 1. Inserting into the ear. If the ear tip feels uncomfortable or it is difficult to insert, place the filter into a smaller size ear tip (6-11 mm) - 3. Changing ear tip size. Please contact us if it is not possible to achieve a proper seal with the supplied ear tip sizes.

### 1. Inserting into the ear

1. Ensure the filter is correctly positioned in the ear tip (Image G, H & I).

2. Grip the tongue of the ear tip between thumb and forefinger (Image B).

3. Pull the top of the ear outward and upward to ease insertion (Image C).

4. Position the tongue at the top of the ear making the product easier to remove (Image D).

5. Gently push and twist the product into the ear until it sits comfortably in the ear canal whilst forming a good seal with the ear canal.

### 2. Removing, Cleaning and Storage

1. Find and grip the tongue of the ear tip (Image D).

2. Remove the product with a slow twisting motion to gradually break the seal with the ear canal.

3. Clean the ear tips after each use. It can be wiped clean with a damp cloth or antibacterial cleaning tissues. We recommend only cleaning the filters if necessary. Do not use alcohol-based cleaning solutions, soaps or detergents.

4. Always store the product in a clean resealable carrying case. Recommended storage temperature range is between -10°C and +50°C.

### 3. Changing ear tip size

1. Remove the filter from the current ear tip by gently squeezing the ear tip just below the filter (Image E).

2. Ensure that the logo in the filter is facing upward (Image A).

3. Push the filter in at an angle to ease insertion (Image F).

4. Press down on the filter until it is correctly positioned in the ear tip (Image G, H & I).

## WARNINGS

• If these instructions and warnings are not adhered to, the protection afforded by the product will be severely impaired.

• Choking hazard - Keep the product out of reach of young children.

• The product must be fitted, adjusted and maintained in accordance with the instructions. Improper fitting can reduce its effectiveness.

• In order to achieve the designed attenuation level, it's important that the outside of the ear tip has an airtight seal with the ear canal.

• Performance of the product may be reduced after contact with sharp objects e.g. by puncturing or cutting (Image K).

• Regularly inspect the product to ensure serviceability (Image D).

• The product may be adversely affected when coming into contact with chemicals or alcohol.

• Sudden or fast removal of the product out of the ear canal can damage the eardrum.

• Suitable hearing protection must be worn at all times in noisy surroundings.

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